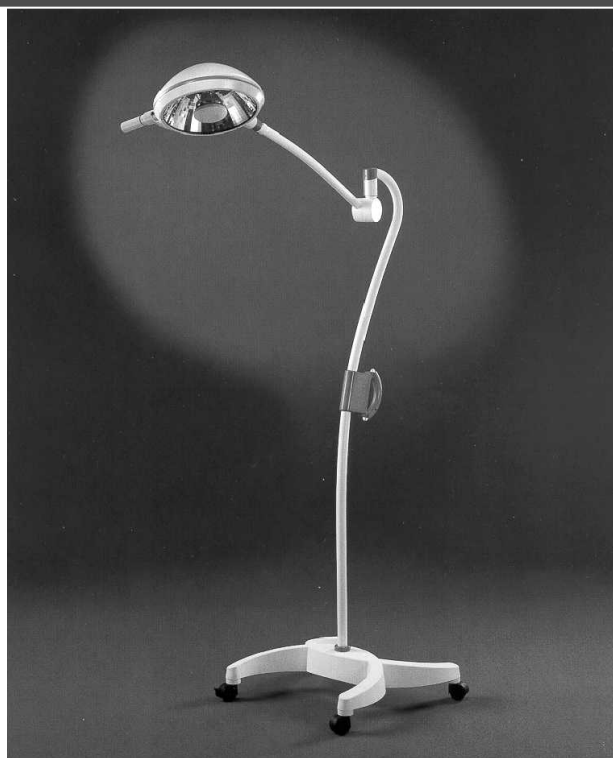
Untersuchungsleuchte ML 301

Examination Light ML 301

Lámpara de exploración ML 301

Lampe d'examen ML 301

Lampada per esami ML 301

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Mounting / Operating and Service Instructions

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© by Gebrüder Martin GmbH Co. KG

This document is under the
copyright protection of Gebrüder
Martin GmbH Co. KG.

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This document must not be
modified, amended, reproduced or
transmitted to a third party without
prior written permission by
Gebrüder Martin GmbH Co. KG.

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In case of technical questions or if
replacement parts are required,
please contact our hotline service.

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We reserve the right to make
changes in design and devices.

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Continuation Part 2

Table of Contents	Page
9.3 Mounting the mobile light	84
9.4 Mounting the light head	92
9.5 Replacing the Spring Arm	96
9.6 Adjusting the spring resistance	110
9.7 Disassembly instructions	112
9.8 Scrapping the lamp	114
10 Maintenance and Repair	116
11 Technical Data of the ML 301	120
11.1 General data	120
11.2 Dimensional drawings	124
11.3 Circuit diagrams of the ceiling-, wall-mounted and mobile lights	127
11.4 Toroidal transformer wiring options	132
11.5 Wiring of the single- phase transformer	135
12 Replacement Parts ML 301 Mobile, Wall, Ceiling	136
12.1 Mounting positions for the ML 301 ceiling-mounted lights replacement parts	141
12.2 Mounting positions for the ML 301 wall-mounted lights replacement parts	141
12.3 Mounting positions for the ML 301 mobile lights replacement parts	142
13 Type Plate / Serial Number	143
13.1 Position of the type plate on the ML 301	143
13.2 Serial number of the ML 301	143
14 Statics Certificate	146
Transfer Certificate - KLS Martin Examination Lights ML 301	150
Drilling Jig for Ceiling or Wall Mounting of the ML 301	155

Continuación Parte 2		Continuation Partie 2		Continuazione Parte 2	
Contenido	Página	Table des matières	Page	Indice	Pagina
9.3 Montaje de la lámpara de pie	85	9.3 Montage des lampes sur pied	85	9.3 Montaggio delle lampade su stativo	85
9.4 Montaje del cuerpo de la lámpara	93	9.4 Montage du corps du luminaire	93	9.4 Montaggio del corpo lampada	93
9.5 Cambio del brazo de suspensión elástica	96	9.5 Remplacement du bras à ressort	96	9.5 Sostituzione del braccio a molla	96
9.6 Ajuste de la fuerza elástica	110	9.6 Réglage de l'effet de ressort	97	9.6 Regolazione della reazione elastica	97
9.7 Advertencias para el desmontaje	112	9.7 Instructions de démontage	112	9.7 Istruzioni di smontaggio	112
9.8 Desguace de la lámpara	114	9.8 Mise à la ferraille du luminaire	101	9.8 Smaltimento della lampada	101
10 Mantenimiento y reparación	116	10 Maintenance et réparation	116	10 Manutenzione e riparazione	116
11 Datos técnicos de la ML 301	120	11 Caractéristiques techniques du luminaire ML 301	120	11 Dati tecnici della lampada ML 301	120
11.1 Datos generales	120	11.1 Caractéristiques générales	120	11.1 Dati generali	120
11.2 Dibujos acotados	124	11.2 Plan côté	124	11.2 Disegno quotato	124
11.3 Esquemas de los circuitos de las lámparas de techo, pared y pie	127	11.3 Schéma des connexions des lampes fixées au plafond, murales et sur pied	127	11.3 Schema elettrico delle lampade a soffitto, a parete e su stativo	127
11.4 Posibilidades de conmutación del transformador toroidal	132	11.4 Câblages possibles du transformateur toroïdal	132	11.4 Possibilità di cablaggio del trasformatore con nucleo ad anello	132
11.5 Cableado del transformador monofásico	135	11.5 Connexion du transformateur monophasé	135	11.5 Cablaggio del trasformatore monofase	135
12 Piezas de repuesto lámpara de pie, pared, techo ML 301	136	12 Pièces de rechange lampes ML 301 sur pied, murales, fixées au plafond	136	12 Parti di ricambio lampade ML 301 su stativo, a parete, a soffitto	136
12.1 Ubicaciones de las piezas de recambio de la lámpara de techo ML 301	141	12.1 Positions des pièces de rechange de la lampe ML 301 fixées au plafond	141	12.1 Posizioni dei pezzi di ricambio della lampada ML 301 a soffitto	141
12.2 Ubicaciones de las piezas de recambio de la lámpara de pared ML 301	141	12.2 Positions des pièces de rechange de la lampe murale ML 301	141	12.2 Posizioni dei pezzi di ricambio della lampada ML 301 a parete	141
12.3 Ubicaciones de las piezas de recambio de la lámpara de pie ML 301	142	12.3 Positions des pièces de rechange de la lampe ML 301 sur pied	142	12.3 Posizioni dei pezzi di ricambio della lampada ML 301 su stativo	142
13 Placa de características / número de serie	144	13 Plaque d'identité / numéro de série	144	13 Targhetta / numero di serie	145
13.1 Ubicación de la placa de características en la ML 301	144	13.1 Position de la plaque d'identité au luminaire ML 301	144	13.1 Posizione della targhetta alla lampada ML 301	144
13.2 Número de serie en la ML 301	144	13.2 Numéro de série luminaire ML 301	144	13.2 Numero di serie lampada ML 301	144
14 Prueba estática	147	14 Documentation statique	147	14 Documentazione statica	147
Protocolo de entrega - Lámparas de exploración ML 301 KLS Martin	151	Rapport de réception - Lampes d'examen KLS Martin ML 301	152	Verbale di consegna - Lampade per esami KLS Martin ML 301	153
Plantilla para taladrar los agujeros para el montaje de la lámpara de exploración ML 301 en el techo / en la pared	155	Gabarit de perçage pour l'installation au plafond / murale de la lampe d'examen ML 301	155	Sagoma di foratura per montaggio a soffitto / a parete della lampada per esami ML 301	155

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Modification service

After having delivered these instructions, modifications and/or supplements of text or drawings were made on the pages indicated in the following.

On the substituted pages, the modified parts are marked by a number in a circle followed by a vertical bar.

The first modification on a page is indicated by ①

Any other modification on the same page (which is carried out on a later point of time) is marked by ② etc., and the marking of the previous modification will be removed.

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Datum / Version	Seite	Agenda
09/07 V. 2.0	64	①
09/07 V. 2.0	72	①
09/07 V. 2.0	74	①
09/07 V. 2.0	76	①
09/07 V. 2.0	78	①
09/07 V. 2.0	80	①
09/07 V. 2.0	88	①
09/07 V. 2.0	92	①
09/07 V. 2.0	92	②
09/07 V. 2.0	96	①
09/07 V. 2.0	102	①
09/07 V. 2.0	102	②

Modification table

Date / Version	Page	Agenda
09/07 V. 2.0	64	①
09/07 V. 2.0	72	①
09/07 V. 2.0	74	①
09/07 V. 2.0	76	①
09/07 V. 2.0	78	①
09/07 V. 2.0	80	①
09/07 V. 2.0	88	①
09/07 V. 2.0	92	①
09/07 V. 2.0	92	②
09/07 V. 2.0	96	①
09/07 V. 2.0	102	①
09/07 V. 2.0	102	②

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1 Introduction

1.1 Preface to these Operating Instructions

The lights bear the CE-sign, i.e. they meet the basic requirements stipulated in the CE directive on medical products.

These instructions for use will provide detailed information as to how the examination lights are put into service and used for the intended purpose.

The manufacturer warrants that these examination lights have been properly tested. In addition, we are prepared to assume continued responsibility for the safety, reliability and proper functioning of the lights as long as all checking and repair work required is carried out by technical staff that has been duly authorised by Gebrüder Martin GmbH Co. KG to perform such work, and in as much as the examination lights are used only for the intended purposes.

Gebrüder Martin GmbH Co. KG will assume no liability whatsoever for any damage that has been caused by improper use of the product!

For safe and continued use of the lights, it is necessary that you use only original KLS Martin replacement parts.

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1.2 Field of Application

These instructions refer to KLS Martin examination lights with the designations:

- ML 301 ceiling
- ML 301 wall
- ML 301 mobile

Observe the correct type of lamp and refer to the corresponding pages for specific instructions.

The other chapters are valid for all lights.

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1.3 Proper Use for the Intended Purposes

The lights may only be used for purposes in accordance with their design:
i.e., for lighting an examination area by surgeons, doctors, or appropriately trained paramedics.

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1.4 Entry Inspection

Please check the light(s) and the accessories for shipping damage and completeness immediately after having received them.

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1.5 Complaints

Damage may only be claimed for if the vendor or forwarding company are immediately informed.

Immediately establish a certificate of damage and send it to the nearest agent of the manufacturer or directly to the manufacturer.

If possible, use the original packages when returning goods.

Enclose the following accompanying documents with the returned goods:

- name and address of the owner
- type and device number
- description of the defect

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1.6 Warranty

For the examination lights, we give a warranty of 12 months, starting from the day they were delivered to the user.

Within this warranty period, our competent service agencies or the manufacturer will eliminate all defects for free which were demonstrably caused by manufacturing faults or faulty material.

Parts subject to wear (e.g. lamps) are excluded from this guarantee.

The general guarantee and warranty terms of **Gebrüder Martin GmbH Co. KG** are valid.

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1.7 Symbols in these Mounting Instructions

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Texts labelled by this symbol and printed in bold italics contain very important information, also for the protection from hazards to human health! You must observe these instructions under all circumstances!

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Texts labelled by this symbol and printed in bold italics contain important information, also for the prevention of material damage! You must observe these instructions under all circumstances!

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Texts labelled by this symbol and printed in bold italics contain comments / notes or hints.

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This circle marks activities which you must carry out.

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This symbol marks references to graphics or chapters.

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2 Safety Instructions

All KLS Martin examination lights are high-quality products that have been designed and manufactured in accordance with technical standards. At the time when they leave the factory, they fully meet all safety-related requirements.

In order to preserve this condition, you as the user are requested to avoid and prevent all actions that may have a negative influence on the safety of the lamps, and to have damaged parts repaired immediately or inform the manufacturer about the damage!

Damaged lights may be a life- or health-threatening hazard for the user and / or the patient! For this reason never use damaged lights!

Handling, activation and deactivation of the lights may only be carried through in accordance with the information provided in these instructions for use!

The mounting of the ceiling or wall mounted lights and the repair of all lamps may only be carried out either by the manufacturer or by a person or company explicitly authorised by the manufacturer!

The ceiling-mounting light may only be installed on a fireproof ceiling.

To move the mobile light on its castors, always use the handle provided on the stand pillar! Using the light head handle (operating handle) for this purpose could lead to spring arm fracture!

It is not allowed to put additional weight on the lamps.

For this purpose, read Part 2, Mounting and Service Instructions.

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3 Initiation

Prerequisite is that the lamp has been correctly assembled and installed by a service technician.

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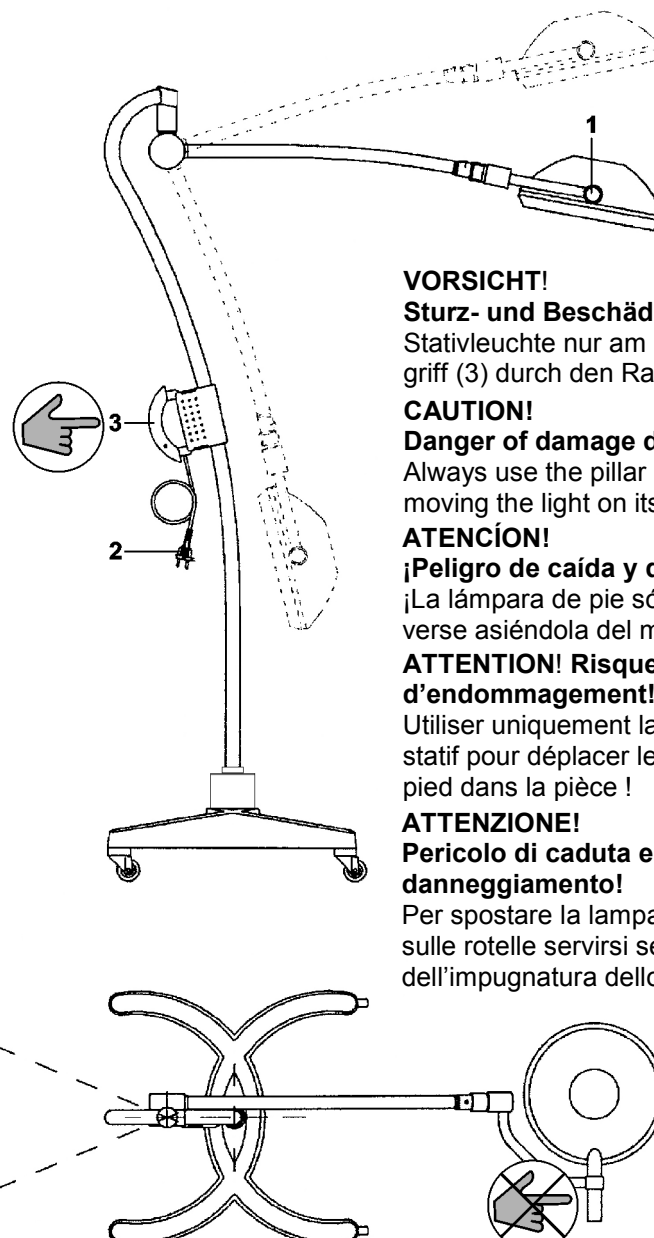
- 1 ON / OFF-switch
on light head handle
- 2 Mains plug
- 3 Handle

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CAUTION!

Danger of damage due to crash!

Always use the pillar handle (3) for moving the light on its castors!

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3.1 Operation of the Lamp

The light emission side of the lamp must be at a distance of at least 60 cm from the body of the patient.

The lamps can be positioned precisely by means of the handle.

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3.1.1 ML 301 D

- Operate the ON / OFF switch (1) on the handle of the lamp.

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3.1.2 ML 301 W

- Operate the mains switch (2) on the wall-mounted cabinet.
- Operate the ON / OFF switch (1) on the handle of the lamp.

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3.1.3 ML 301 S

**Danger of damage due to crash!
Always use the pillar handle (3) for moving the light on its castors!**

- Connect the mains plug (2) to a socket which has been installed in accordance with the regulations.
- Ensure the correct voltage and type of current (230 VAC, 50 / 60 Hz)!
- Compare with the instructions on the type plate.
- Operate the ON / OFF switch (1) on the handle of the light.

The locking rollers of the stand base make it possible to lock the lamp into place.

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- Switch off the light.

Hot components!

- **Let the light head cool down before replacing the bulb**

- Release the lampholder on the underglass side by turning it to the left.
- Carefully slew the lampholder downwards.
- Draw the bulb out of the holder.
- Place the new bulb into the holder.
- Only install original replacement bulbs supplied by the manufacturer!

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When using other bulbs the light efficiency will be lower, the colour temperature will vary and the heat radiation will be increased!

Never touch the lamp bulb with your bare fingers.

- Carefully slew the lampholder upwards.
- Insert the lugs of the lampholder into the corresponding cutouts and lock the lampholder into place by turning it to the right. The holder must sensibly engage.

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Personnel training!

Cleaning and disinfecting must be carried out by trained personnel.

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5.1 Cleaning

WARNING - Electric shock!

Prior to cleaning, switch off the light, pull the mains plug and secure the unit against unintentional reconnection.

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Never allow cleaning fluid to penetrate into the unit.

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To prevent damage to plastic parts, do not use scouring agents, alkaline cleaning agents or agents containing acid or alcohol.

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- **Pull mains plug and allow unit to cool down.**

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- **Wipe the light's surfaces with a slightly moistened cloth. If necessary, add some mild soap solution (washing-up liquid).**

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5.2 Disinfection

WARNING - Electric shock!

*Prior to disinfecting, switch off
the light, pull the mains plug
and secure the unit against
unintentional reconnection.*

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*Never allow disinfecting fluid to
penetrate into the unit.*

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CAUTION – Damage to the unit!

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*To prevent damage to the light
housing:*

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- *do not use disinfectants
containing (more than 60%)
alcohol;*
- *disinfect the unit only in cold
condition;*
- *do not use high-alkaline
solutions, acids or pure
alcohol.*

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Teil 2 / Part 2 / Parte 2 / Partie 2 / Parte 2

Montage- und Service-Anweisung

Mounting and Service Instructions

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6 Introduction

6.1 Preface to these mounting and service instructions

The work on the ML 301 lamps (assembly, disassembly, maintenance and repair) requires good electrical and mechanical skills.

Therefore, this type of work may only be carried out by service technicians authorised by the manufacturer of the lamps!

As lamp manufacturer, we strongly recommend that you – the service technician – get acquainted with

- the functioning method and operation of the lights and
- the work to be carried through

by means of these mounting and service instructions before beginning to work.

Therefore read all of these mounting and service instructions carefully and observe the safety information under all circumstances!

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6.2 Duty of certification

After service work, the service technician must fill in a certificate (transfer certificate).

This certificate must contain

- the date when the work was carried through,
- the type and scope of work,
- the name of the company and
- the signature of the service technician.

The certificate can be found on the last pages of these instructions (⇒ page 134).

If a service is not carried out by the authorised service technician of the manufacturer, the components which have been modified or repaired must also contain the identification mark of the service technician.

- In case of first installation or repair, copy the transfer certificate
- Fill in the certificate and send it to the address mentioned on the bottom.

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WARNING – electrical shock!

In case of damage or faulty assembly of the electrical equipment (e.g. lines, plug-in connections, components), the support arm system can carry electrical voltage. In this case, there is danger of a lethal electrical shock!

- *Therefore, this equipment must be assembled carefully and correctly!*
- *Never use damaged electrical components or components with damaged electrical insulation (e.g. mains cables, switches, fuses etc.)!*

Do not repair such components but replace them immediately by new ones!

- *The ceiling and wall mounted lights are connected to the building's power supply by means of a stationary cable junction, the mobile light by means of a mains cable with a plug.*
- *For the ceiling and wall mounted lights, a disconnecting switch (e.g. a circuit breaker) must either be present in the building's power supply system or it must be installed so that all poles can be separated from the light's electrical circuit at the same time.*

For this purpose, a switch for the clearances and creeping distances given in IEC 328 (VDE 0630) or which is UL-listed must be used.

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8 Mounting Instructions

The descriptions and graphics in this chapter refer to mounting of the ceiling and wall mounted lights and of the mobile lights and to the process of electrical installation.

Prerequisite is that

- you have read chapter **7 Safety Instructions for the Service Technician**,
- you read the entire chapter 8,
- the position of the light on the ceiling or on the wall, respectively, has been determined and that
- the building's electrical installation has been carried through with reference to the mounting position.

You have to assume the responsibility for the proper execution of the mounting work!

- For the load data for ceiling or wall mounting, refer to the following table.

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Load data for ceiling mounting		
bending moment	vertical weight	dowel extraction
96 Nm	100 N	590 N

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Load data for wall mounting		
bending moment	vertical weight	dowel extraction
96 Nm	80 N	470 N

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9.1.1 Drilling the fastening holes

Observe the safety instructions in chapters 7 and 8!

- Remove the drilling jig on the last page of these instructions.
- By means of this jig, mark the positions for the fastening holes on the ceiling in such a way that the dimension drawn in fig. 9-1-1 results between the bore hole (2) in the ceiling plate and the outlet opening (1) for the ceiling power supply lines.
- Drill the four fastening holes in accordance with the diameters and the lengths of the fasteners to be used.

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9.1.2 Mounting the ceiling plate

- Fasten the ceiling plate (1) with the fasteners (2). Make sure that the plate touches the ceiling in a horizontal plane.
- Tighten the fasteners with a torque wrench in accordance with the manufacturer's instructions.

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Make sure that no bare wires protrude from the terminals; otherwise, there is danger of an electric shock or a short-circuit!

- **Therefore, this equipment must be installed carefully and correctly!**

- By means of the transformer wiring, check whether the transformer is correctly wired for the mains voltage.

If necessary, correct the transformer wiring (chap. 11.5).

- Connect the power supply lines to the connecting terminals of the transformer.

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9.1.4 Assembly of the Soffit

- Slide the soffit (1) over the ceiling tube until it touches the ceiling on all sides and secure it in this position by means of the fastening ring (2).
- Screw all three grub screws into the fastening ring.

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9.1.5 Mounting the Swing Arm with Spring Arm

- Unscrew the recessed countersunk head screw (9) from the covering lid (8) and remove the covering lid.
- Slide the swing arm with spring arm (4) over the pivot of the ceiling tube (1) until the bore hole (2) is again visible. Hold the swing arm in this position.
- Insert the lug of the lug disk (6) into the bore hole (2) and install the sealing ring (7) in the groove (3).
- Check the sealing ring for proper fit.
- Carefully and with slight pressure, slide the covering lid (8) over the swing arm tube in such a way that the connector pin is inserted into the female plug of the ceiling tube.
- Fasten the covering lid (8) in the bore hole (5) with the recessed countersunk head screw (9).

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WARNING – danger of injury!

- Without assembled lug disk, the sealing ring is opened if the swing arm moves.
- If the sealing ring is not correctly fit into the groove, it can fall out if subjected to a load.

This causes the swing arm to fall down which can lead to severe injuries.

- It is therefore indispensable that you assemble the lug disk and check the sealing ring for proper fit.

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WARNING – electric shock!

In case of damage or faulty assembly of the electrical equipment (e.g. lines, plug-in connections, components), the support arm system can carry electrical voltage. There is danger of a lethal electric shock!

- Therefore assemble this equipment carefully and correctly!

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You have now finished the mounting of the support arm system and the electrical installation of the ceiling light.

Mounting of the light head and adjustment of the spring arm are described under

9.4 Mounting the light head and

9.5 Adjustment of the spring resistance.

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There are two different types of wall bracket available:

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- Wall light with 115 VAC wall bracket
- Wall light with 230 VAC wall bracket

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- Remove the drilling jig (on the last page of these instructions).
- By means of this jig, mark the positions for the fastening holes on the wall in such a way that the dimensions drawn in fig. 9-2-1 result between the floor, the bore hole (1) of the wall bracket and the outlet opening (2) for the wall power supply lines.
- Drill the two fastening holes in accordance with the diameters and the lengths of the fasteners to be used.

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- Remove the two plastic coverings (2) and (4) from the wall bracket.
- Fasten the wall bracket (1) by means of the fasteners (3). Make sure that the wall bracket plate touches the wall in a vertical plane.
- Tighten the fasteners according to the manufacturer's instructions by means of a torque wrench.
- Replace the two plastic coverings and check them for proper fit.

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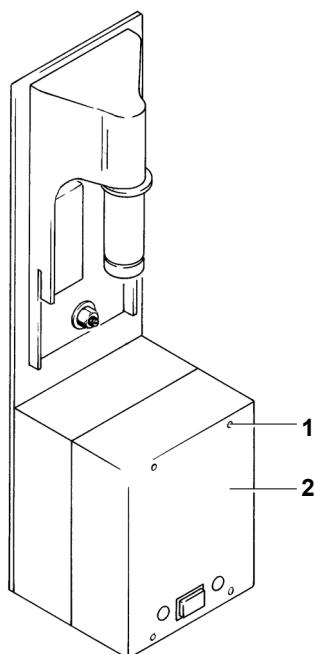
WARNING – electric shock!

In case of damage or faulty assembly of the electrical equipment (e.g. lines, plug-in connections, components), the support arm system can carry electrical voltage. There is danger of a lethal electric shock.

Make sure that no bare wires protrude from the terminals; otherwise, there is danger of an electric shock or of a short-circuit.

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- **Therefore, this equipment must be assembled carefully and correctly!**
- Unscrew the four screws (1) of the transformer box to the point that you can open the lid (2) of the box downwards.
- By means of the transformer wiring, check whether the transformer is correctly wired for the mains voltage.

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Prerequisite for beginning to mount the wall lights is that you have read the chapters

- 7 **Safety Instructions for the Service Technician and**
- 8 **Mounting Instructions.**

If you have not yet read these chapters, please do so now and return to this chapter afterwards.

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9.2.2.1 Drilling the fastening holes

The person responsible for mounting the light is also responsible for selecting the correct fasteners in compliance with the load specifications provided in chapter 8 (see “load data” table) and for carrying out the mounting work properly.

For lightweight walls, we recommend using a counter-plate (not part of the scope of delivery).

Load data:

The load data for wall- and ceiling-mounting lights are available from the “load data” table provided in chapter 8. The prescribed regional safety factors must be included as well.

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Depending on wall design (wall structure), two or four dowels can be used for fixing the bracket housing in place. For concrete walls, it is sufficient to use the two heavy-duty anchors (HST M10x90 / 10, through-type) supplied with the product (to be mounted diagonally, top and bottom).

- **Remove the drilling jig (on the last page of these instructions).**
- **By means of this jig, mark the positions for the fastening holes on the wall.**
- **Drill two / four mounting holes according to the diameter and length of the fasteners to be used.**

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- 1 Protective conductor connection
- 2 Opening for power supply cable
- 3 Floor

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9.2.2.2 Mounting the wall bracket

- From the rear, run the power supply cable (1) through the opening of the wall bracket housing (2).
- Using the fasteners (5), fix the wall bracket housing (6) in place, making sure that the housing attaches flatly and vertically to the wall.

- Tighten the fasteners (5) with a torque of 45 Nm.

- Connect the power supply lines (1) to terminal (7).

Brown (L1)
Blue (N)
Green / yellow (PE)

- From the rear, run the 13.4-V supply line (1) through the opening of the wall bracket housing (3).

The wall bracket (3) must be fitted to the bracket housing (6) with three hexagon socket head cap screws DIN 912 M8x25 A2 (8, 9).

- Tighten one of the hexagon socket head cap screws (8) in the lower section of the wall bracket (3), using a torque of 30 Nm.

- Tighten two hexagon socket head cap screws (9) in the upper section of the wall bracket.

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- Therefore, this equipment must be assembled carefully and correctly!

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- Connect the two 13.4-V supply lines (1) from the power converter to the wall bracket terminal (2) provided in the housing.

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- Slide the first cover of the wall bracket housing (4) in place on the wall bracket (3) from above.

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- Put the second cover of the wall bracket housing (5) in place on the wall bracket (3) from below.

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- Tighten the cross-recessed head screw (6) on the cover of the wall bracket housing (5).

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9.2.2.4 Mounting the swing arm with the spring arm

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- Without assembled washer (3) and tab washer (4), the sealing ring (5) will open if the swing arm moves.
- If the sealing ring (5) is not fit correctly into the groove, it can fall out if subjected to a load.

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This causes the swing arm (2) to fall down which can lead to severe injuries.

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- It is therefore indispensable that you assemble the washer (3) and tab washer (4) and check the sealing ring (5) for proper fit!

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- Remove the cross-recessed head screw (7), then pull cap (6) forward and remove it upwards.

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- Slip swing arm with spring arm (2) over the neck of the wall bracket.

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- Insert tab washer (4) into bore (1) and mount circlip (5).

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- Put cap (6) in place and secure it with the cross-recessed head screw (7).

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Prerequisite for beginning the mounting of the mobile light is that you have read the chapters

- **7 Safety Instructions for the Service Technician and**
- **8 Mounting Instructions.**

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If you have not yet read these chapters, please do so now and return to this chapter afterwards!

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- Insert the pivots of the two anti-static rollers (1) (with braking mechanism) through the ring cable sockets (2) of the PE lines into the stand bases (3) until they stop.

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- Insert the other two rollers (without braking mechanism) into the other stand bases until they stop.

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WARNING – electrostatic charging

If the roller pivots are not in contact with the PE line, this can lead to electrostatic charging of the stand and, under certain circumstances, to a discharge on the user or the patient!

- **The PE line must therefore be installed correctly.**

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9.3.2 Attaching the coloured foil

- Clean the eye-shaped indentation (4) of dust and fluff.
- Remove the protective foil (3) from the adhesive side of the coloured foil.
- Place the coloured foil (2) in the indentation and press it on evenly.
- Remove the transparent protective foil (1) from the coloured foil.

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9.3.3 Assembly of the stand pillar

- Unscrew the wing screw (6) with the split washer (5) from the stand pillar (1).
- Insert the stand pillar into the opening in the stand base in such a way that the lug of the stand base (3) is locked into the cutout in the tube (2) and the tube can no longer be turned.
- Slide the wing screw with the split washer through the bore hole (4) of the supplementary weight and fasten the stand pillar in this manner.

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CAUTION – damages to the stand

If the lug safety device is not locked into place and the stand pillar fastened subsequently, the stand will fall over!

- Therefore, you must lock the lug safety device into place and tighten the wing screw with the split washer.

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9.3.4 Assembly of the plastic half-rings

- Insert the two plastic half-rings (2) onto the transformer housing (3) and fasten them by means of the recessed countersunk head screws (1).

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CAUTION – damages to the unit

If the plastic half-rings are not inserted, the stand will fall over!

- **Therefore, insert the half-rings, screw them tight and then check them for proper fit!**

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9.3.5 Mounting the spring arm

- Unscrew the recessed countersunk head screw (2) from the covering lid (1) and remove the lid.
- From the bottom, insert the spring arm pivot (10) into the pillar (7) until it stops and hold it in this position.
- Place the disk (4) over the pivot and insert the sealing ring (3) into the groove (9) of the pivot.
- Connect the female (8) with the male plug (5).
- Place on the covering lid (1) and fasten it in the bore hole (6) by means of the recessed countersunk head screw (2).

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WARNING – danger of injury

- *If the disk is not assembled, the sealing ring will be opened if the spring arm is moved.*
- *If the sealing ring is not properly fit into the groove, it can fall out if subjected to a load.*

This causes the spring arm to fall down which can lead to severe injuries.

- *It is therefore indispensable that you assemble the disk and check the sealing ring for proper fit!*

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You have now finished mounting the mobile light.

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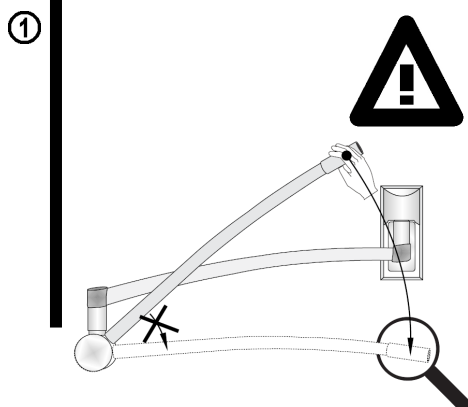
Mounting the light head to the spring arm is the same for the ceiling-mounted, wall-mounted and mobile lights!

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The spring arm will be damaged if it is pulled beyond the lower / upper stop.

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- Grab the spring arm at the front end (not at the rear end) and slowly pull it downward until it reaches the stop.

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If the spring arm is pressed down and then released, it can spring up and cause injuries!

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- Make sure that no one else is within the swivelling range of the spring arm while you are mounting the light head!

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- Draw the protective cap out of the spring arm opening.

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- Check the black rubber ring (1) for firm seating, it must be positioned firmly behind the four knobs in the plastic sleeve (4).

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- Slide the bushing (4) onto the spring arm in such a way that the groove (3) in the bushing aligns with the slot (2) in the spring arm.

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- Insert the pivot (9) of the light head cardanic suspension into the spring arm opening.

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
In the plug-in connection, the male and the female plug are integrated. This ensures the power supply of the light head.

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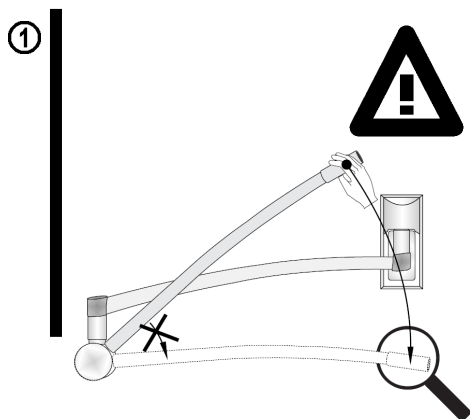9.5 Replacing the Spring Arm

This section describes how to replace the spring arm ML 301.

CAUTION – Damage to the device:

If the spring arm is pulled over the lower / upper limit stop, the spring arm will be damaged.

- Hold spring arm by the front end (**not the rear end**) and slowly pull it downwards until it reaches the limit stop.

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WARNING – Danger of injury

The spring arm can snap upwards when released and cause injuries!

- Ensure that there are no other persons in the swiveling area of the spring arm when replacing it!

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WARNING – Electric shock!

Before starting to replace the spring arm, separate it from the electric circuit, unplug mains connector and secure it against being switched on again.

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The spring arm is now fully mounted on the cantilever / stand. Now the lamp body must be mounted.

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The lamp body with the new spring arm on the cantilever / stand is now fully mounted.

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The spring resistance of the spring arm is adjusted in identical manner for the ceiling-mounted, wall-mounted and mobile lights.

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Adjust the spring resistance in such a way that the light head stops in every position.

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- Move the light head downwards approx. 20° from the horizontal position until the bore hole (1) is on top and a slotted screw head is visible in the bore hole.

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- Insert a screwdriver for slotted screws into the bore hole.

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If the spring arm is lowered, the spring resistance is insufficient:

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- Rotate the adjusting screw clockwise until the spring arm with the light head remains suspended in any position.

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If the spring arm rises, the spring resistance is too strong:

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- Rotate the adjusting screw counterclockwise until the spring arm with the light head remains suspended in any position.

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- After the adjustment, remove the screwdriver from the bore hole.

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The lamp is now completely mounted and adjusted!

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For initiation of the lamp, read
3.1 Operation of the lights.

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9.7 Disassembly instructions

The light may only be disassembled (e.g. for service work or for shutdown of the lamp) by a service technician who has been authorised by the lamp manufacturer!

WARNING – electric shock

If the lamp is disassembled while switched on, there is a danger of touching parts which carry current! This can lead to a lethal electric shock!

- *Therefore, switch off the power supply of the light before any disassembling work; disconnect all poles of the lamp from the building's power supply!*
- *Make sure – e.g. by erecting a warning sign – that the lines cannot be energised unintentionally during disassembly!*

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WARNING – danger of injury

The spring arm has a strong spring resistance!

If the light head is not removed in the uppermost spring arm position, the spring arm springs up and can cause severe injuries.

- *The light head may only be disassembled if the spring arm is in the uppermost position!*

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The support arm system and the ceiling plate or wall bracket, respectively, are heavy!

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During disassembly of the safety contrivances and fasteners, these parts can fall down and cause severe injuries!

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- *Before disassembly, secure these parts against falling!*

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The lamp is always disassembled – in accordance with chapter 9.6 Disassembly instructions – in reverse order as compared to 9 Mounting the lights!

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- *To remove the securing segment (⇒ 9.4 Mounting the light head), use tweezers or a small, pointed screwdriver.*

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The light does not contain any material which is hazardous to the environment. After correct disassembly and separation of the different types of material (plastic, aluminium, copper, steel, glass), it can be recycled.

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10 Maintenance and Repair

WARNING – Electric shock!

Prior to performing any checks or maintenance work, de-energize the light (pull mains plug) and secure it against unintentional reconnection.

All KLS Martin examination lights must be checked for the following at regular intervals:

Every six months:

- defective lacquer coating
- defective plastic parts (cracks)
- suspension arm system deformation

Annually:

- Supply main undamaged?
- Is the mains switch undamaged and does it function safely?
- Mains fuse okay?
- Mechanical connections okay?
- Smooth, but not too smooth operation of links?
- Brake force adjusted properly – does the lamp stop in every position?
- Correct stability (of the mobile light)?
- Check fixing screw on underside of stand base; retighten if necessary.
- Check and grease securing segment.
- Lamp, lamp mount, reflector and underglass okay?

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Type of unit: B

Supply voltage under load:

ceiling-mounted light: 13.8 V
(at the transformer
output)

wall-mounted light: 13.8 V
(at the transformer
output)

mobile light: 13,2 V_{eff}
(at the luminous element)

Mains cable length

(mobile light): 4 m

Weight

- ceiling-mounted light: 10 kg

- wall-mounted light: 8 kg

- mobile light: 15 kg

Electrical system
according to: VDE / IEC

Colour:

RAL 9010
(hardwarestatus HW 06 or higher)

RAL 9002
(up to hardwarestatus HW 05)

Ambient conditions

Transport and storage:
-10°C to 50°C (-4°F – 122°F) at
a rel. humidity of 20% to 90%
(non-condensing)

Operating temperature:
5°C to 40°C (23°F – 104°F) at a
rel. humidity of 30% to 80%
(non-condensing)

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Light head

area of operation: 60 - 140 cm
 swivel radius
 - ceiling-mounted light: 1800 mm
 - wall-mounted light: 1800 mm
 - mobile light: 960 mm

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Halogen bulb

voltage, output: 12 V, 50 W
 service life of the
 lamp: 2000 h

Illuminance
 at 0.8 m distance: 45000 lux

colour temperature: 4300 K

light field Ø: 180 mm

colour rendition
 index $R_{a(1-8)}$: 93 - 96

light efficiency
 (luminous
 efficiency): 290 lm/W

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When needing a replacement transformer, the supply voltage must be indicated on the order or the transformer must be wired on-site in accordance with the local supply voltage.

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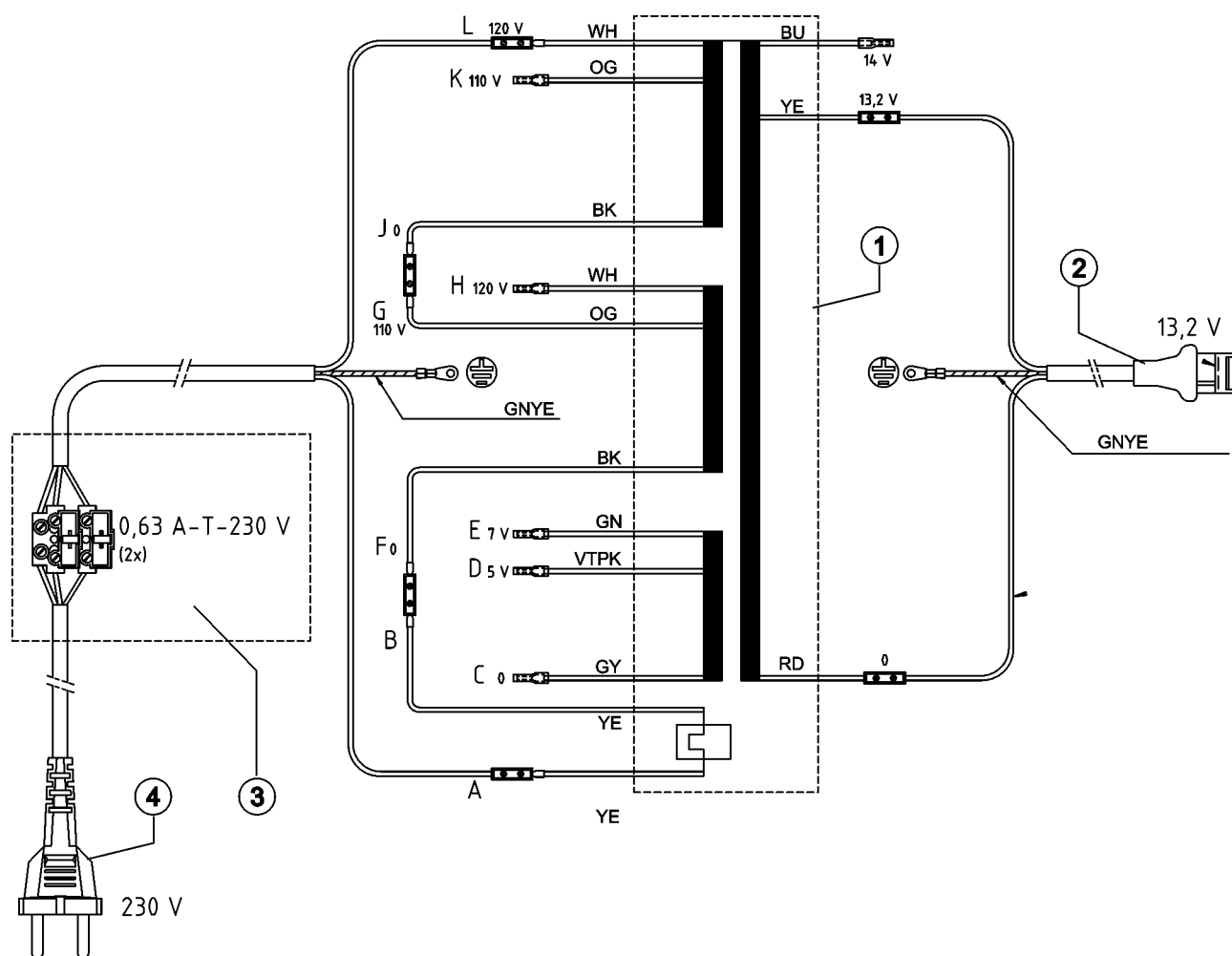
11.5 Wiring of the single-phase transformer

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12 Replacement Parts ML 301 Mobile, Wall, Ceiling

Pos.	Designation	Number	Order no.
1	Spring arm (mobile)	1	08-060-01-20
2	Stand base cpl. with casters	1	08-084-00-42
3	Stand pillar lacquered, compl. w. 230-V electronic transformer (up to HW 04)	1	08-084-00-01
4	Handle halves cpl. with two 3 x 12 screws and a plug	2	08-063-00-25
5	Screw set for transformer installation, consisting of a cable clip, two 3 x 10 screws, two 3 x 6 screws	1	08-084-00-55
6	Cover cap, red, mobile cantilever	1	08-051-00-21
7	Power outlet, 3 poles (spring arm)	1	08-076-00-10
8	Plug, 3 poles (light head)	1	08-076-00-07
9	Caster set (4) for stand base	1	08-653-27-04
10	Half of adapters, red with two 2.2 x 9.5 screws	2	08-051-00-50
11	Spring arm (wall / ceiling)	1	08-653-22-04
12	Ceiling cantilever and spring arm „S“, „wing“	1	08-653-24-04
13	Wall bearing with cantilever and spring arm	1	08-653-25-04
14	Wall bracket with covering	1	08-092-00-20
15	Securing segment (cardan joint / spring arm)	1	08-075-01-10
16	Safety sleeve with brake	1	08-084-00-51
17	Braking screw for safety sleeve	1	08-053-00-39
18	Fuse (0,5 AT / slow-blow at 230 V)	1	08-084-00-12
19	Fuse (1,25 A at 115 V)	1	08-084-00-45
20	Holding ring for soffit for ceiling tube Ø 42	1	08-053-00-35
21	Soffit Ø 360x105x42 mm	1	08-053-00-37
22	Soffit cover Ø 400x2x42 mm (not shown)	1	89-700-06-04
23	Transformer ML 301 D / W (ceiling / wall) 115 / 230 V, ML 301 S (mobile) 115 V	1	08-087-06-04
24	Elektronic transformer for ML 301 S, 230 V up to HW 04 and ML 301 W, 230 V (HW 07 or higher)	1	08-087-07-04
25	Toroidal transformer stand, from HW 05	1	08-087-08-04
26	Halogen bulb (cold-light mirror 50 W)	1	86-903-01-04
27	Socket with screws (without cable and socket holder)	1	08-063-00-26
28	Rubber profile red	1	08-077-00-10
29	Cover	1	08-063-00-30
30	Underglass Ø 287	1	08-090-00-20
31	Switch of the light head	1	08-084-00-05
32	Power outlet, 3 poles (ceiling tube)	1	08-076-00-08
33	Plug, 3 poles (swing arm)	1	08-076-00-09
34	Stand pillar compl. w. toroidal transformer (100–247 V) (HW 05 or higher)	1	08-087-01-04
35	Fuse (0.63 AT / slow-blow at 230 V) for mobile version (HW 05 or higher)	1	08-034-00-07
36	Filter glass w. socket for cold-light mirror with counter-ring	1	08-051-00-51
37	Socket compl. w. cables and cap	1	08-063-00-31
38	Mains switch, 1-pin, black, 33x25mm	1	08-084-00-06
39	Reflector cone and reflector	1	08-062-00-20

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12.1 Mounting positions for the ML 301 ceiling-mounted lights replacement parts

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12.2 Mounting positions for the ML 301 wall-mounted lights replacement parts

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12.3 Mounting positions for the ML 301 mobile lights replacement parts

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13 Type Plate / Serial Number

13.1 Position of the type plate on the ML 301

The picture shows the position of the type plate.

13.2 Serial number of the ML 301

<u>ML</u>	<u>301</u>	<u>00</u>	<u>00</u>	<u>99</u>	<u>D</u>	<u>136</u>
product type		hardware version	software version	year of construction	model	number of item

product:	ML ▲	KLS Martin examination light
model:	D ▲	ceiling-mounted version
	W ▲	wall-mounted version
	S ▲	mobile version

With each new year of construction, the numbers of the items start again with 001.

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14 Statics Certificate

The statics certificate must be filled in before the wall or ceiling lights are mounted!

- The construction's carrying capacity must be planned, checked and confirmed by a structural engineer.
- The relevant regional regulations for construction must be observed.
- In case of faulty drilling, for example, tapping of a reinforced-concrete construction, the responsible structural engineer must be consulted since the static load distribution in the ceiling can get out of balance.

Acceptance certificate:

This is to confirm that the carrying wall / ceiling and the anchorage of ML 301 are safe and capable of carrying the load.

Project:

Place:

**Signature / stamp
(structural engineer /
construction authority):**

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Transfer Certificate – KLS Martin Examination Lights ML 301

The most important data for each mounted lamp are recorded by Gebrüder Martin GmbH Co. KG in a tracing system. Please return the certificate after you have filled it in.

The technician must confirm correct assembly and testing by signature; correct transfer must be confirmed by the customer.

Address of customer:

Location of use or installation:

Lamp / article no.:

Serial no.:

☐ first assembly repair ☐
☐ modification disassembly ☐

date of transfer:

1. Scope of delivery

 delivery of the lamp according to the delivery note ☐

 Mounting / Operating and Service Instructions ☐
2. Sight check

 labelling and type plates ☐

 sliding contacts ☐

 light visibly okay ☐
3. Mechanical check

 ceiling tube adjusted ☐

 free torsion guaranteed ☐

 free-moving height adjustment ☐

 free-moving movement of light head ☐

 brakes adjusted ☐
4. Sight check

 according to IEC 601-1/EN 60601-1 ☐

 protective conductor contact resistance
 < 0.1 ohm ☐

 – between equipotential bonding strip on the
 ceiling tube and all metallic components
 of the light combination ☐

 – between the terminal block (PE connections)
 of the control box and all metallic parts of
 the control box tangible from the outside ☐

 insulation resistance > 20 mohm ☐
5. The light is ready for operation ☐

if not, measures taken

 information to Technical Service director ☐
6. Function check

voltage in the transformer (primary side)

desired: 230 / 115 VAC actual: _____ V

illuminance at 1 m distance _____ klux

 focussing ☐
7. Remarks

 (signature of customer)

 (signature of erecting engineer)

 (date)

1. customer 2. erecting engineer 3. Martin ServiceCenter

 Gebrüder Martin GmbH & Co. KG • Postfach 60 • D-78501 Tuttlingen
 Telephone: +49 74 61 / 7 06-0 • Telefax: +49 74 61 / 7 06-193

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Stuckenbrock Medizintechnik GmbH
Martin Nederland/Marned B.V.
1270 AG Huizen · The Netherlands
Rudolf Buck GmbH
KLS Martin UK Ltd.